

USER MANUAL



Set up and demonstration of the
Feline Urinary Catheter Model



Male Feline Urinary Catheter Model

Revision A:
Male Feline Urinary
Catheter User Manual

Product SKU:
SKU: F100-000

Last Updated:
11 September 2023



Parts List

Equipment Required:

- Desk or Table
- Syringe
- Bladder Fill Solution
- Vaseline
- Paper Towel
- Sponge

Spare Parts Available to Order

- Replaceable Suture Block
- Penis/Bladder
- Bladder Casing
- Pelvis bone
- Access Door
- Locking Sliders
- Fill Tube & Cap
- Desk Clamp
- Knurl Screws

For all other parts or repairs, please contact Holsim through our website:
www.holsim.co.nz




The Product





The Prep

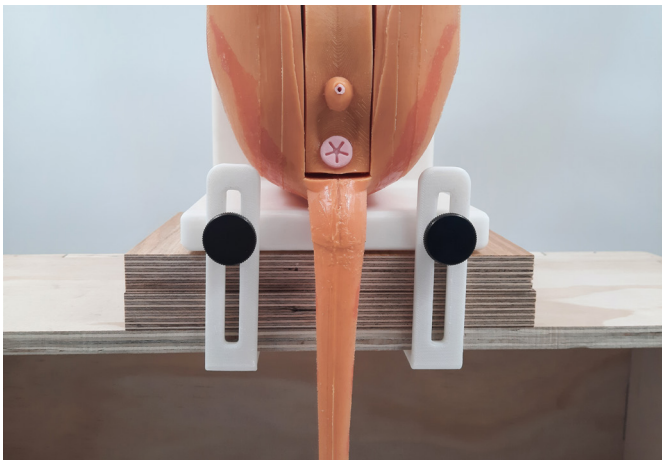


1 tighten table clamps

- Slide the table clamps upwards until they sit against the underside of the table or desk.
- Tighten the knurl screws so the model is secured in place.



2 remove suture pad

- Pull the suture pad off of the cat.
- You should be able to see the penis.

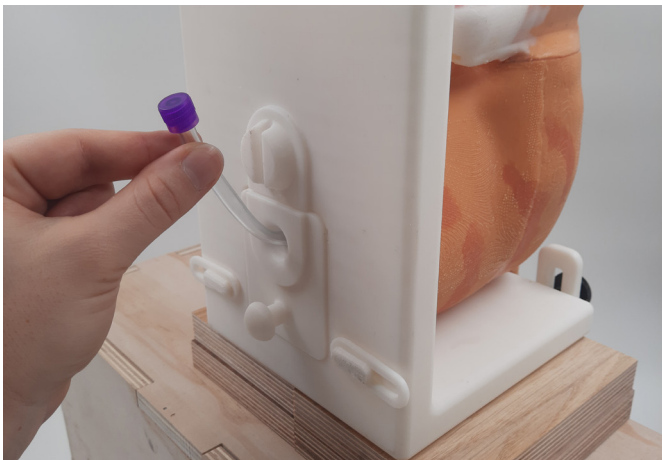

3

unlock internal case sliders

- Move the access door sliders outwards to the open position to unlock the access door.



4

unclip fill tube

- Unclip the fill tube from the holder at the back of the stand.




The Prep




5 remove internal case

- Pull the internal bladder case out of the cat.

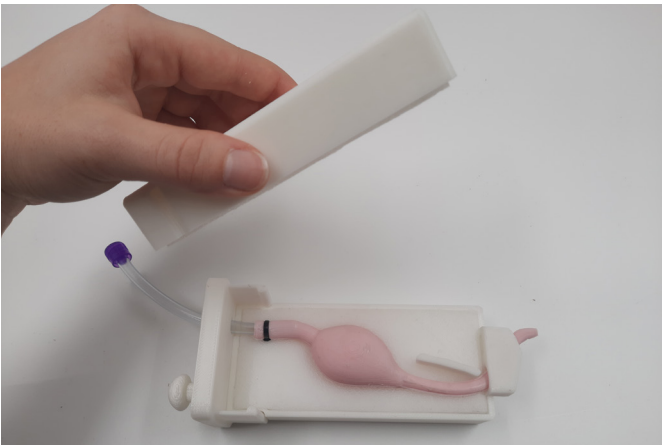

6 open the case

- Slide one side of the bladder casing off the access door.

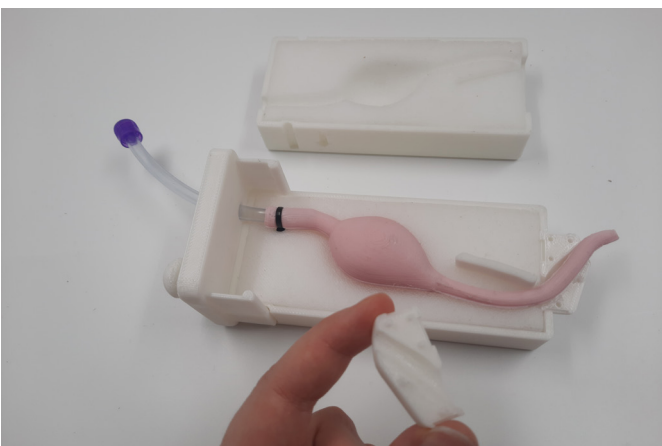


7 remove foreskin block & check parts

- Ensure bladder and pelvis bone are laying in the correct positions in the foam surround.
- The fill tube should be laying in the access door hole and the penis should be exiting through the foreskin block
- Ensure there are no punctures or damage to the silicone parts.




8 apply lubricant

- Open the two sides of the foreskin block and apply a light layer of Vaseline or other lubricant into the main channel.

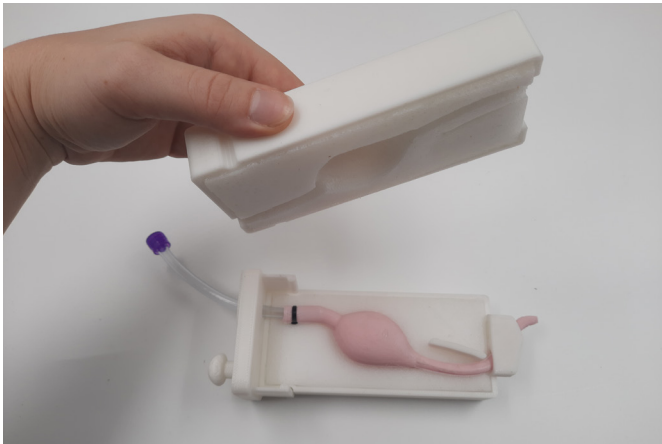

9 reassemble foreskin block & casing

- Slide the bladder casing back onto the access door, being careful keep the foreskin block aligned.

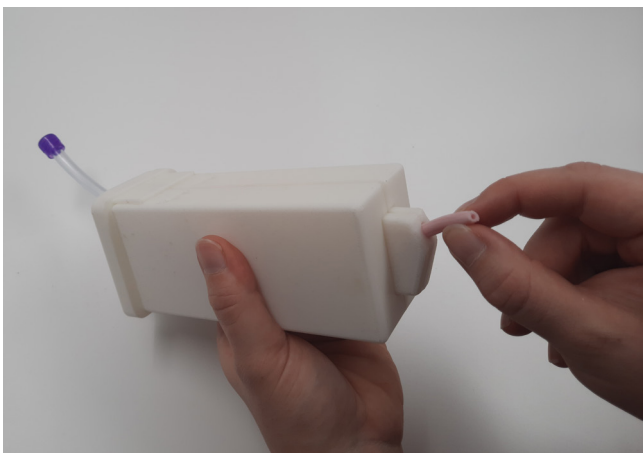

10 gently stretch the penis




The Prep




5 insert casing into cat

- Slide the casing back into the stand and into the cat, ensuring it is the correct orientation upwards.
- Apply a light dusting of baby powder to the inside of the cat and outer case if needed to prevent sticking.

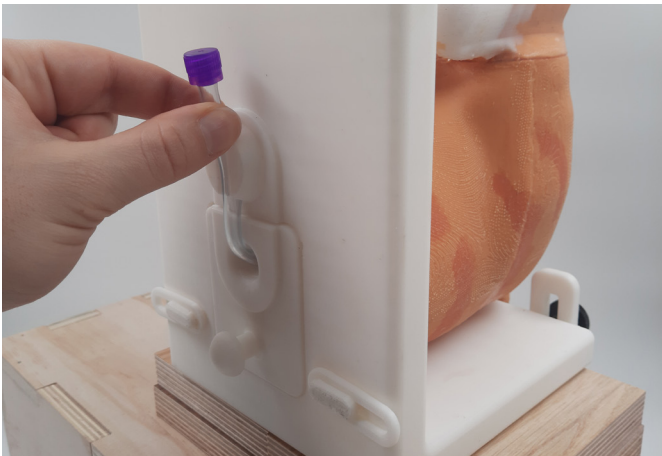


6 clip fill tube in place

- Clip the fill tube into the holder at the back of the stand.

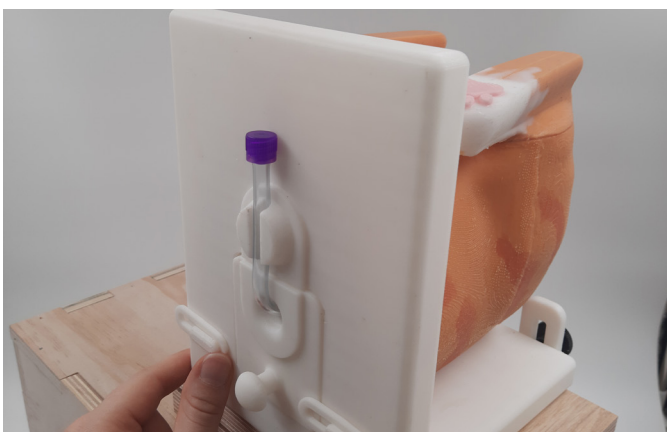

7 lock internal case sliders

- Move the access door sliders inwards to the locked position

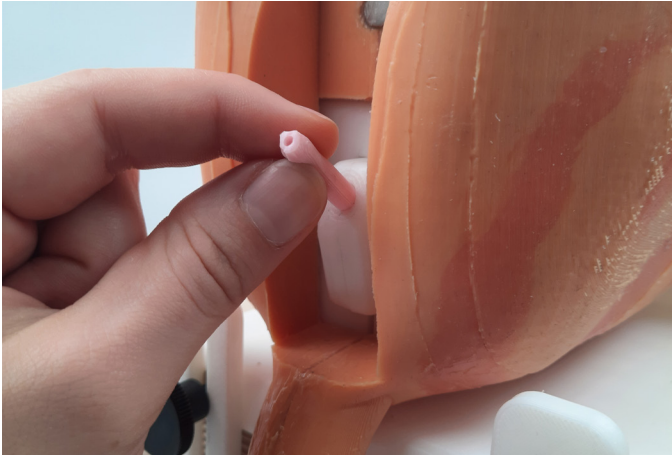

8

gently stretch penis again



9

align penis with hole in suture block & click in place

- Push the suture block into the backside of the cat. It should snap into place.




10

check penis

- opening of the foreskin in the suture block.
- If it is too far down, pull it off and stretch the penis a little more until it's near the opening of the foreskin.



The Prep

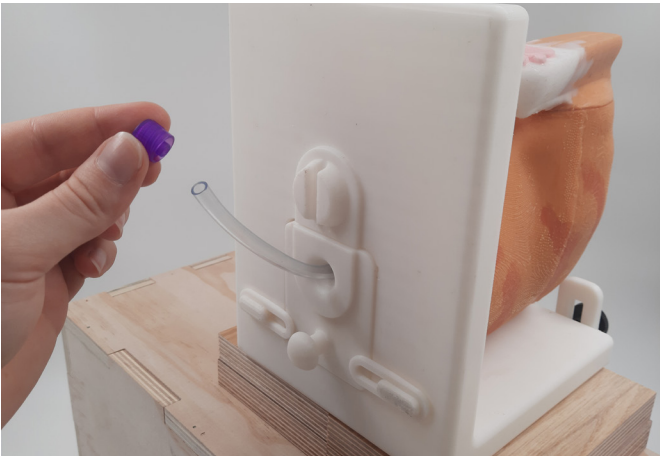


5 remove fill tube cap

- Remove the cap from the fill tube at the back of the stand.



6 fill bladder

- Take your syringe and fill the bladder with 20ml of desired solution. This will fill the bladder.
- Any overflow will come out of the penis.

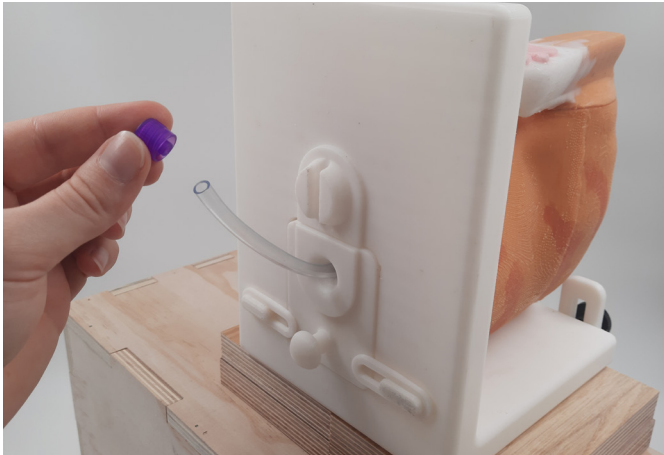


8 replace fill cap

- Replace the cap and tuck the fill tube back into the holder at the back of the stand.



9 position model

- Sit the model at the edge of your table or desk

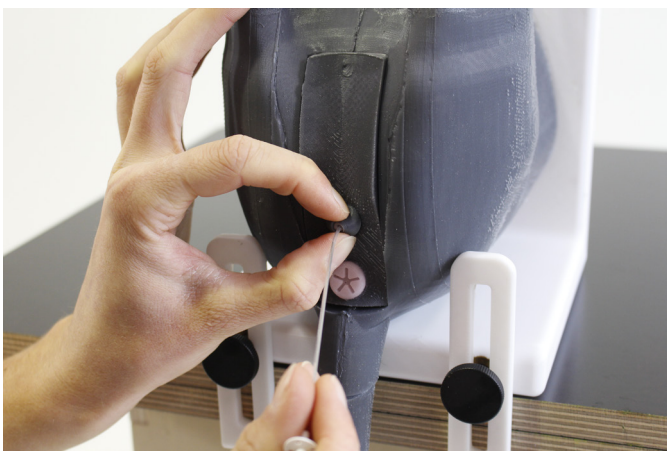

9 perform operation

- Now the operation may be performed as normal.
- During the operation, ensure the catheter is inserted slowly and carefully to prevent puncturing the bladder wall.
- Ensure catheter is stitched to the correct area on the suture block.
- Once the operation has been performed successfully, cut any leftover stitching out of the suture block to allow it to be reused in future.



Post OP



5

remove suture block


- Remove suture block from backside of cat model.



6

remove foreskin block


- Remove the foreskin block.
- Open and wipe Vaseline away with a clean cloth.
- Wash with warm soapy water and dry with a paper towel.

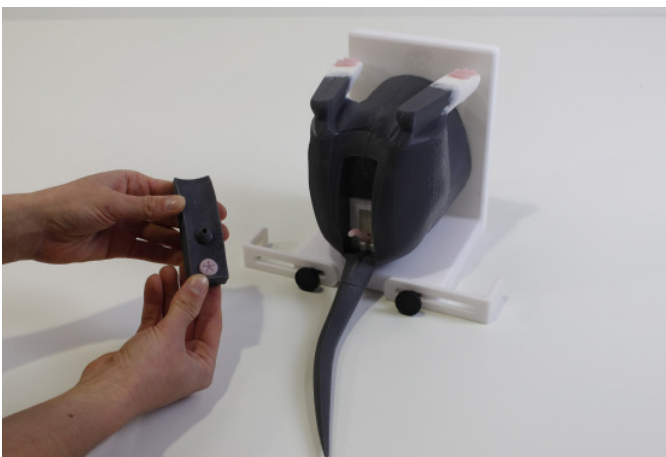

7

wash suture block


- Wash suture block with warm soapy water to remove any Vaseline.
- Pat dry with a paper towel and allow to dry.

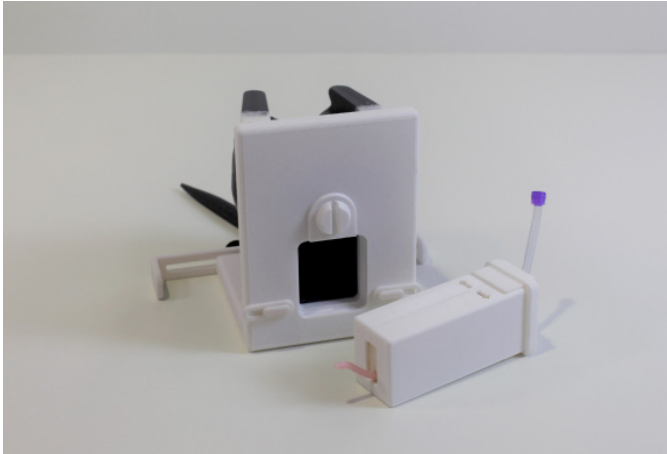

8

remove bladder casing

- Slide internal bladder casing out of the cat and open.

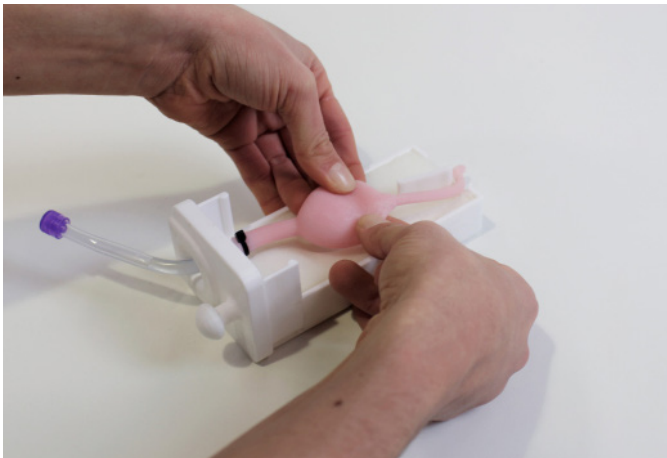

9

examine penis & bladder

- Check to see if the bladder and penis are in tact after the operation.
- Gently stretch the silicone to examine for puncture holes from the catheter.



10

drain bladder

- Whilst holding the bladder over the sink, use the syringe to drain the bladder.
- This can be done by sucking the water out with the syringe or pushing air through the bladder with the syringe.
- Allow to dry before putting it away.



Post OP



5 clean and dry cat

- Gently wipe down the cat with a wet sponge to remove any debris or dust.
- Pat dry with a paper towel



6 replace bladder casing

- Replace internal drawer with bladder inside once dry.



7 replace fill hose

- Replace cap onto fill hose and tuck hose into clamp on the back of the stand.



8

storage

- Ensure cat is fully cleaned, dried and assembled for the next use.
- Store away from direct sunlight in a plastic container to keep dust and debris from settling onto the silicone skin.



9

damaged parts

- If parts are damaged, worn or need replacing; please get in touch with Holsim via our website

www.holsim.co.nz



Holsim

34D Somerset Street,  07 839 5018
Hamilton, New Zealand  info@holsim.co.nz